

MALLEODYN S3
DISMOUNTABLE ANKLE BRACE

BETTER IN MOTION



SPORLASTIC HOLISTIC TREATMENT CONCEPT

ALL-ROUND CARE

Particularly the ankle is exposed to high stress. This can lead to acute injuries and chronic overload reactions of ligaments, tendons, bones and joint cartilage, especially during sport, but also in everyday life. Braces can support an early functional and phase-oriented healing process.

SUPPORTS AND BRACES HELP

According to a representative **Allensbach patient survey**¹ on the effectiveness of orthopaedic aids, **66 %** of the wearers of support/braces, who previously had to regularly take pain-relieving medication were now able to **dispense with analgesics** either completely or partially as a result of using the aid.

With the **MALLEODYN S3**, SPORLASTIC offers a dismountable ankle brace with different degrees of stabilisation, which supports an early functional and phase-oriented recovery.

References:

- 1) Representative survey by the Institute of Public Opinion Research in Allensbach on behalf of Eurocom e.V. 2014: "Use and effectiveness of orthopaedic aids"





Functional aids and appliances



Special exercise programmes



Helpful, tried and tested training equipment

INCLUDES PHYSIOTHERAPY TO GO DIRECT BENEFIT FOR YOUR PATIENTS

SPORLASTIC ankle braces, in combination with medical and physiotherapy measures, support a holistic treatment concept that includes the original aspects of the underlying indications.

Offer your patients true additional benefits:

A QR code on selected products takes patients with a smartphone or tablet to important product information, as well as guided therapeutic exercises designed to support faster recovery. You can also use our patient brochures with their additional interesting content.

And the best part is that your patients have universal access to this vital information and to exercises and can carry them out by themselves.




MALLEODYN S3

DISMOUNTABLE ANKLE BRACE FOR DYNAMIC MOBILISATION

The innovative ankle brace can be easily dismantled in 3 steps according to the course of therapy. In the acute phase, it stabilises, relieves and cools the ankle according to the RICE rule (rest, ice, compression, elevation).

Due to its flexible inner-shoe-like construction and easy handling, the **MALLEODYN S3** can be used for secondary prevention after the 3 dismantling steps.

INDICATIONS

- Conservative/post-operative
 - All indications for which stabilisation or limitation of movement of the ankle with subsequent functional mobilisation of the injured or damaged structures is required
 - Acute ankle injuries
 - Acute capsular ligament instabilities and ruptures
 - Restriction of talus feed and tilt
 - Chronic capsular ligament instabilities
 - Supination trauma
 - Ankle sprains
 - Prevention of ligament instabilities
-
- **Article No.** 07841
 - **Colour:** 
 - Please specify R or L



FULLY ASSEMBLED

In the acute phase – cooling pad included

Fully assembled, the brace can be used optimally according to the RICE rule. The enclosed cooling pad, combined with a laterally stabilising splint and a figure 8 strap, relieves and stabilises the ankle in the acute phase. After a one-time individual basic adjustment, the brace is placed in neutral position via the open ventral access with just two clicks.

DISMOUNTABLE IN 3 STEPS

1



1. DISMOUNTING STEP

After dismounting the cooling pad

After the acute phase, stabilisation is achieved with anatomically shaped splints including the additional stiffening lateral splint (orange) in pro- and supination direction. The limitation of talus feed and plantar flexion is achieved by the figure 8 strap positioned under the metatarsus.

2



2. DISMOUNTING STEP

After dismounting the side splint

Due to the bi-elastic material, the brace adapts optimally to the anatomy of the ankle and allows further mobilisation in both pro- and supination direction. In addition, anatomically shaped sewn-on splints with deep heel mounts and the figure 8 strap stabilise the ankle joint.

3



3. DISMOUNTING STEP

After dismantling the figure 8 strap

After the dismantling of the figure 8 strap, the brace permits maximum mobilisation for the ankle joint according to the course of therapy. The quick and intuitive application of the brace still takes place with just two clicks.



FULLY DISMOUNTED

Long wearing time and secondary prevention

The high-quality materials with antibacterial effect make the innovative brace a supporting aid for sports or for an active everyday life. The dismantled brace can offer safety and stabilisation in the acute phase as well as dynamic guidance for prevention.

Maximum breathability

Anatomically shaped side splint –
easy to dismount with one **click**



Leg fastener – quick to put on
with one **click**

Bi-elastic inner pad
for pressure-free ankles



Foot fastener – easy to close
with one **click**

Dismountable
figure 8 strap

Antibacterial effect
of the interior padding
with carbon finish



Attached **cooling pad** for
therapy-compatible positioning



Physiotherapy to go
via QR code on attached card



SPORLASTIC ORTHOPAEDICS – BETTER IN MOTION

SIZE

MALLEODYN S3

shoe size

1	35	–	40
2	41	–	47



SPORLASTIC GERMANY

SPORLASTIC GmbH

Medical Products

Weberstraße 1 • 72622 Nürtingen • Germany

PO Box 14 48 • 72604 Nürtingen • Germany

Phone +49 7022 705181

Fax +49 7022 705113

info@sporlastic.de • www.sporlastic.de

SPORLASTIC QUALITY



Quality management system
DQS-certified according to
DIN EN ISO 13485:2016,
Reg. No. 069305 MP2016



MITTELSTAND
GLOBAL
HEALTH MADE
IN GERMANY

SPORLASTIC DIGITAL



www.facebook.com/sporlastic



www.youtube.com/sporlastic



www.instagram.com/sporlastic_official